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EN
PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: metakrylowe resiny (bis-GMA, UDMA, TGDMA), inorganiczny 62%, pigments, photoiniciator, inhibitor.

INDICATIONS FOR USE
The RAINBOW FLOW light-curing composite (colour: orange, yellow, green, blue, violet, pink, dentine) is designed for:
- filling cavities in deciduous and permanent teeth as the first layer in Class I and Class II cavities
- sealing fissures

- dental splinting and also as a protective marking of the enamel border when setting up temporary splints and retainers.

RAINBOW FLOW (orange, yellow, green, blue, violet, pink) is also designed for marking the mouth of root canals.

RAINBOW FLOW (dentine) is also intended for the restoration of anterior and lateral teeth and for the attachment of orthodontic brackets and the protection of the mouth of root canals.

When using composite materials, it is not advisable to employ preparations containing eugenol, as this may disturb the composite polymerization process.

INSTRUCTIONS FOR USE

Before treatment, dip the included applicator in spirit (ethyl alcohol) and then place it on a syringe.

Filling cavities, sealing fissures:

Cover the treated cavity/fissure with a bonding system according to the manufacturer's instructions.

Fill the cavity with a composite layer of no more than 2 mm in thickness. Then, cure with a dental curing lamp for 20 seconds with light intensity of at least 550 mW/cm². It is recommended that the distance between the light source of the dental curing lamp and the restorative surface should not exceed 2-3 mm. If necessary, apply another layer of the material.

RAINBOW FLOW can be worked and polished directly after polymerization using standard methods for composite materials.

Marking the mouths of root canals (after permanent filling of the root canal)

Apply a thin layer of the composite to the canal according to the manufacturer's instructions. Then, apply a composite layer of spirit 2 mm in thickness and cure with a dental curing lamp for 20 seconds. The composite material helps to accurately locate the mouths of closed root canals during the next visit.

After squeezing the material out of the syringe, release the plunger to prevent further outflow of the material.

Immediately after using the preparation, withdraw the syringe plunger and close the package with the original cap in order to avoid polymerization of the preparation in the syringe.

The product is intended for multiple use, while the applicator is for single use only. Reusing the applicator may pose a risk of secondary infection.

CONTRAINDICATIONS
Do not use in patients who are hypersensitive to any of the ingredients.

PRECAUTIONS:
This product can cause eye irritation in the uncured form. In case of contact, rinse with plenty of water and contact a doctor immediately.

If swallowed, drink plenty of water. Do not induce vomiting. Contact a doctor immediately. It can cause allergic skin reaction.

Use rubber dams, disposable gloves, safety goggles and protective clothing when working with this product.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in a tightly closed original container below 25°C.

Protect from direct sunlight. Reseal after use.

Keep out of the reach of children.

The expiry date is on the direct packaging.

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