ENDO aspirator

SMART SYSTEM

Wireless system for drying root canals

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Device description:

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EN

Wireless **Endo-Aspirator Smart System** is a suction system for drying root canals operating independently from the unit. The system uses the vacuum absorption technique. The device is equipped with the next generation air pump, which creates an appropriate centrifugal force, draining away excess rinsing fluids and impurities resulting from the preparation of the root canal.

Endo-Aspirator Smart System offers three modes of operation (low, medium, high). The selection of the appropriate operating mode depends on the quantity and density of the irrigation fluids and the power required to effectively dry the canal.

For better user experience, **Endo-Aspirator Smart System** is equipped with a readable LED display with suction level, battery level and operating time indication.

Endo-Aspirator Smart System is a compact and comfortable to use device that helps to achieve better clinical results faster during endodontic treatments.

The system allows for thorough drying of the root canal, which is essential for the effective sealing and filling of the root canal.

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Device operation:

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Turning the device on: press the on/off button. When turned on, the device will default to low speed. In order to select medium speed, press the on/off button twice, while to select high speed – press the on/off button three times.

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Turning the device off: press the on/off button for 2-3 seconds.

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Battery charging: to charge the device battery, use the charger included in the package.

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The charging time for a fully discharged device battery is up to 60 minutes.

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The working time of the device after a full battery charge is up to 70 minutes.

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Use of the system:

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1

Screw the bottle adapter (4) onto the bottle for liquids (5).

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2

Place the suction device (1) and the bottle with the adapter (4-5) in the stand (8).

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3.1

Connect the suction device (1) to the bottle with the adapter (4-5) using the short hose (2). Screw the luer-lock thread end to the suction device (1).

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3.2

Press the connector located at the other end of the short hose (2) onto one of the tabs of the bottle adapter (4).

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4

Connect Endo-Aspirator PRO (7) with the long hose (3) – use the luer-lock thread end.

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5

Connect Endo-Aspirator PRO (7) to the bottle with the adapter (4-5) using the long hose (3). Press the connector located at the end of the hose onto the free tab of the bottle adapter (4-5).

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Press / naciśnij

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8

Insert the applicator (6) into the prepared root canal and start drying.

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9

When you have finished drying the root canal, turn the suction device off (1) by holding the on/off button for 2-3 seconds.

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10

Disassemble all the components of the kit. Clean the suction device and Endo-Aspirator PRO – follow the instructions for use.

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11

Remove the outer cap and the inner filter cover from the suction unit (1) and check the contamination level of the filter (11). If necessary, replace the inner filter (11).

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The bottle with the adapter (4-5), hoses (2-3) and applicator (6) are for single use. Endo-Aspirator PRO (7) is a re-usable instrument suitable for sterilisation in an autoclave.

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