PL

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EN

### PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:**  
MTA+ powder: calcium oxide, oxides of: silicon, iron, aluminium, sodium, potassium, bismuth, magnesium, zirconium; phosphate of calcium.

### INTENDED USE

The product is intended for dental treatment as a material for filling and remineralization of root canals. The compound cures in 6 to 10 minutes after mixing with water. The high content of calcium ions remineralizes the tooth tissue. The silicon and calcium compounds support tissue regeneration after pulpulation and intracanal resorption. MTA+ is especially recommended in:

- root canal wall perforation
- intracanal resorption
- filling of the posterior root apex
- direct coating of the pulp
- pulp amputation
- treatment of teeth with interrupted root development.

MTA+ is not suitable as a standalone permanent root canal filling.

### INSTRUCTIONS FOR USE

Cut off the MTA+ liquid pipette tip. Move the entire contents of the MTA+ powder glass vial with 1 to 2 drops of the MTA+ liquid on a mixing plate.

Mix for ca. 30 until the compound reaches a consistency of moist sand. If the compound mix is too thick or too brittle, add one more drop of the liquid (or deionized water) and mix again.

Exposure to moisture, including blood, will reduce the curing time. Once mixed with the MTA+ liquid (or deionized water) the mixed dose must be used within 10 minutes. The compound cures and hardens afterwards. Uses the contents of the MTA+ liquid container within 12 months after opening.

### CONTRAINDICATIONS

Do not use in patients who are hypersensitive to any of the ingredients. Do not use in inflammatory conditions. Acidic pH values disrupt setting of MTA+.

### PRECAUTIONS

The product can cause eye damage. Irritating to mucous membranes of the oral cavity and skin. In case of contact with water and contact a doctor immediately. If swallowed, do not induce vomiting. Drink plenty of water. Contact a doctor immediately. Use rubber dams, disposable gloves and safety goggles when working with the product.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

### STORAGE

Store in a tightly closed original container below 25°C. Protect from moisture. Keep out of reach of children. The expiry date is on the direct packaging.

### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

### PACKAGING

Glass vial containing 0.14 g of the MTA+ powder; pipette containing 1 ml of the MTA+ liquid.

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