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Root canals filling and sealing material
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: epoxy-amino resin containing zirconium oxide and calcium phosphate, excipients.

INTENDED USE

Permanent root canal filling of the permanent teeth using gutta-percha points. Permanent root canal filling of the permanent teeth using root canal points. The material can be used for all techniques of the root canal sealing. It is perfect for thermic methods.

Contrasting in X-ray imaging.

INSTRUCTIONS FOR USE

SYNTEX sealing paste is provided in a syringe dispensing two pastes in the ratio 1:1. Before the first use, squeeze a small amount of the product directly from the syringe to ensure even paste dispensing and then place the mixing tip onto the syringe. Product can be also mixed without a mixing tip. To do this, squeeze equal amounts of pastes directly on the mixing paper and mix using any tool until obtaining the uniform color.

Prepare, rinse and dry the root canal prior to application. Place a small amount of SYNTEX on the tip of a Lentulo needle. Slowly move the needle towards the apex at the lowest possible micro-engine speed. Remove the needle from the canal using the minimal engine speed. You can also apply SYNTEX paste to a previously disinfected and dried point. Place it in the canal with a pumping movement. Additional points may be placed in the canal or it may be filled with the lateral or vertical condensation of gutta-percha. If the pulp chamber is contaminated with the sealer, use isopropyl alcohol to remove the remains of non-polymerized preparation. The product setting time is about 3 hours and it complies with EN ISO 6876 Root canal sealing materials. A shorter setting time can result in polymerization shrinkage and finally cause the filling to leak. If a repeated endo-treatment is needed, the sealer can be removed with Canal Clean or Methylene Chloride.

CONTRAINDICATIONS

Do not use SYNTEX in patients with hypersensitivity to the product components.

PRECAUTIONS

Use a coffer-dam while working with the product. Use protective gloves, goggles and clothes. Avoid contact of a single paste or non-polymerized material with the skin, mucous membranes and eyes. Contact with a single paste or non-polymerized material causes serious eye damage and may cause an allergic skin reaction.

In case of eye contact: rinse thoroughly with plenty of water for at least 10 minutes. Seek medical advice. In case of contact with the skin or mucous membranes, rinse with plenty of water. In case of irritation or rash, seek medical advice. If swallowed, do not induce vomiting. Rinse the mouth with water. Seek medical advice.

Avoid pushing the product through the root apex as it can cause irritation of the periapical tissues. Work under X-ray and endometer guidance.

STORAGE

Store in a sealed package in a ventilated room at 2-25 °C Protect from sunlight. Keep out of reach of children. The expiry is placed on the product label. Once the container is open, the expiry date doesn't change if the container is securely closed after each use.

When closing the syringe, avoid mixing the pastes from the syringe with the pastes remaining on the closing cap (white with yellow). This can result in material polymerization and opening of the syringe will be impossible.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe containing 10 g of the preparation, mixing tips, mixing papers.

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