

# **BEST-CORD NANO**

Knitted Retraction Cord

CE 1011

Knitted Retraction Cord impregnated with aluminum chloride hexahydrate.  
Used in retraction of gingival tissues around the tooth for improving the results of dental impressions and hemostasis of the gingival margin.

COMPOSITION: Polyester/Polyamide conjugated yarn.  
Impregnated with Aluminum Chloride Hexahydrate 0.50mg ± 0.10mg/inch.

INSTRUCTION FOR USE:

- Cut the impregnated cord to the required length using edge placed on the cap.
- Apply the cord tightly to the cervix of the tooth and press carefully into the sulcus by means of a retraction cord packer.
- Leave the cord within 5 minutes in the sulcus.
- Remove the cord directly before taking the impression.

CONTENTS:

- Dispenser with 254 cm (100 inch).

STORAGE:

- Close the cap after use and store in a dry place.

PRECAUTION:

- For dental use only.
- Keep out of the reach of children.
- Do not use in patients with a history of allergy to aluminum chloride hexahydrate.
- Irritating to skin and eyes.
- Do not use after the expiry date stated on the packaging.



PPH CERKAMED Wojciech Pawłowski  
37-450 Stalowa Wola, Polska  
ul. Kwiatkowskiego 1  
Tel/fax 0048 15 842-35-85