

# RUBBER-DAM cord

Dental dam stabilizing cord



READ THE FOLLOWING INSTRUCTION BEFORE USE  
PRODUCT FOR DENTAL USE ONLY

## USE

To stabilize dental dam place the cord in the interdental spaces between the exposed tooth and the one protected with dam.

4 cm long of cord is sufficient to stabilize the dental dam. After placement, the length of the cord can be adjusted by snipping with scissors.

The use of watersoluble lubricant facilitates the cord fixing in interdental spaces.

## STORAGE

Keep in the original packaging, below 25 degrees Celsius.

Do not expose to solar radiation.

Close tight after the use.

Keep out of reach of children

The expiry date is printed on the direct package.

## PRECAUTIONS

This product contains latex which may cause allergic reactions.


## PACKAGE WASTE DISPOSAL

Used packaging must be disposed of responsibly or returned to the manufacturer.

## PACKAGING

Dispenser containing 1,2 m of cord

Available sizes: fine, medium, thick.

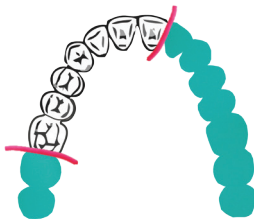

PPH CERKAMED Wojciech Pawłowski  
37-450 Stalowa Wola, Polska  
ul. Kwiatkowskiego 1  
Tel/fax 0048 15 842-35-85