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INTENDED USE

The protective visor is used during dental treatment. It reduces the risk of infections that are transferred by air and protects against the effects of lesser harmful particles during a procedure.

METHOD OF ASSEMBLY

Remove the foil carefully from the box, avoid causing scratches and contact with items likely to cause damage. Connect the endings of the longer side with the clips from both sides of the visor.

Unfold the spectacle frame and using the clips, fix the foil to the spectacle frame.

The foil can be individually adjusted to suit the user.

Do not fold the spectacle frame with the visor foil attached; take off the foil before folding.

NOTICE: Visors are made of thermoplastic material. To shape the visor to a desired shape, soak in hot water beforehand and then shape as required.

MAINTENANCE

To maintain the visor, remove foil from the spectacle frame. Use Fog Buster for Protective Visor, and next wipe surface of the foil using Fog Buster Tissues for Protective Visors.

It lets keep good transparency of Protective Visor and enables safety work. Sheet can be reused. If any scratches or blurring occur as result of use, discard the foil and use another sheet of replaceable foil.

Manufacturer does not take any responsibility for using the visor in a manner not in accordance with instructions or intended use.

CONTENTS OF SET (variants possible)

1. COMFORT VISOR – spectacle frame with metal clips
+ set of exchangeable foils
2. COMFORT LIGHT VISOR – spectacle frame with plastic clips
+ set of exchangeable foils
3. VISOR ATTACHED to corrective eye glasses – set of foils with attached clips for fitting to corrective eye glasses.

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