PL

TCAdent

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TCAdent

Preparation for local cauterisation of mucous membranes

PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE FOR DENTAL USE ONLY**INGREDIENTS:** Trichloroacetic acid 35%, Ph Eur purified water, aroma.**INDICATIONS FOR USE**

TCAdent is used to cauterise local pathological changes and minor bleeding in the oral cavity. The healing effect of this preparation is achieved by chemical coagulation of pathologically altered tissue. The product promotes healing of granulating lesions.

INSTRUCTIONS FOR USE

1. Thoroughly dry the affected area and isolate it (with e.g. vaseline).
2. Soak the tip of the attached applicator in the preparation, get rid of any excess. Rub the altered tissue until it becomes white.
3. Rinse with clean water.

CAUTION! Thoroughly rinse the preparation out, otherwise it may inhibit the setting of impression materials.

If the lesions do not disappear after first use, repeat the procedure several times every 2-4 weeks until their complete disappearance. This product is reusable, but the applicator is intended for single use only. Repeated use of the applicator can pose a risk of secondary infections.

CONTRAINDICATIONS

Do not use in patients who are hypersensitive to any of the ingredients. Do not use this product in children.

PRECAUTIONS

Apply only on the affected area. This product can cause eye irritation. In case of contact with eyes, rinse with plenty of water. If swallowed, do not induce vomiting and drink plenty of water. Contact your doctor immediately. Use gloves, safety goggles and protective clothing when working with the product.

STORAGE

Store in tightly closed original container below 25°C. Keep out of the reach of children. The expiry date is on the immediate packaging. After opening of the packaging, the expiry date does not change, providing it is tightly sealed after each use.

DISPOSAL OF PACKAGING

Used containers should be disposed of or returned to the manufacturer.

PACKAGING

Product available in a dropping bottle containing 10 ml of preparation, with disposable applicators.

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