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Gel for staunching bleeding  
**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
 PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** aluminum chloride 25%, water Ph.Eur., excipients.

**INTENDED USE**

Product ALUSTAT GEL is intended for use during dental treatment to staunch slight gingival bleedings.

Aluminum chloride contained in the product acts vasoconstrictively on tiny blood vessels, as a result the gingival swelling and slight bleedings decrease.

**INSTRUCTIONS FOR USE**

Before the treatment, immerse the included applicator in alcohol (ethyl alcohol) and then place it on the syringe.

Apply the product directly on the bleeding spot, next rinse it with clean water.

Repeat if necessary. Keep on the bleeding site for up to 5 minutes. Product intended for multiple use except the applicator which is for single use only. Multiple use of the applicator may cause the risk of reinfection.

**CONTRAINDICATIONS**

Do not use the product for patients with hypersensitivity to the product's components.

**PRECAUTIONS**

It is irritating to the mucous membrane of the oral cavity, skin and eyes. In case of contact rinse immediately with copious amount of water and seek medical advice.

In case of consumption do not induce vomiting. Drink copious amount of water. Seek medical advice. Use disposal gloves and protective glasses while working with the product.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

**STORAGE**

Store in original packaging at a temperature below 25 °C. Store out of reach of children.

Expiry date is placed on the main container.

Once the container is open, the expiry date doesn't change if the container is tightly closed after each use.

**HANDLING THE PRODUCT CONTAINER**

Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**

Syringe of 5 ml or 10 ml, dispenser, connector, disposal applicators.

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