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EN
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

COMPOSITION: o-phosphoric acid 36%, gel base, dyestuff.

INTENDED USE

BLUE ETCH product is used for etching dentine and enamel prior to composites, fissures and cavity seals.

PREPARATION OF THE PRODUCT BEFORE USE (concerning variant 10 ml and 50 ml)

1. Unscrew the white cap from the dosing syringe containing etchant.
2. Replace white cap with attached connector.
3. Unscrew the blue cap from the connector and replace it with dispenser – smaller syringe attached to the product.

4. Push down the plunger of dosing syringe and pump required amount of gel into the dispenser, but no more than 2ml.
In case of difficulties with pumping the product to the dispenser, move the plunger of the syringe several times. Thanks to that preparation will get runny because of its thixotropic properties.

5. Unscrew the filled dispenser and screw attached applicator onto the top of applicator needle with blunt end). The product is ready for use.

6. Leave the connector screwed on the dosing container and protect it with the blue cap.

INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator into the spirit (ethyl alcohol) and then place it onto the syringe. Apply a thin layer of the etchant onto the dried surface. Etch no longer than 60 seconds. Rinse with strong air-flow and water (for about 15-20 seconds). Do not dry too much.
In case of enamel impurities repeat the process.

Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS
Do not use the product for patients with hypersensitivity to the product’s components.

PRECAUTIONS
May cause severe skin burns and eye damage. In case of contact rinse immediately with copious amount of water and seek medical advice.
In case of consumption do not extract emesis. Drink copious amount of water. Seek medical advice.
Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and protective clothing.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in original packaging at temperature below 25°C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn’t change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed or returned to the manufacturer.

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UK
BEFORE USING THE PRODUCT PLEASE READ THE INSTRUCTIONS CAREFULLY FOR DENTAL USE ONLY

COMPOSITION: orthophosphoric acid 36%, gel base, dye.
INTENDED USE
The product is used for etching dentine and enamel prior to composites, fissures and cavity seals.

PREPARATION OF THE PRODUCT BEFORE USE (concerning variant 10 ml and 50 ml)
1. Unscrew the white cap from the dosing syringe containing etchant.
2. Replace white cap with attached connector.
3. Unscrew the blue cap from the connector and replace it with dispenser – smaller syringe attached to the product.

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5. Unscrew the filled dispenser and screw attached applicator onto the top of applicator needle with blunt end). The product is ready for use.
6. Leave the connector screwed on the dosing container and protect it with the blue cap.

INSTRUCTIONS FOR USE
Before the treatment immerse attached applicator into the spirit (ethyl alcohol) and then place it onto the syringe. Apply a thin layer of the etchant onto the dried surface. Etch no longer than 60 seconds. Rinse with strong air-flow and water (for about 15-20 seconds). Do not dry too much.
In case of enamel impurities repeat the process.
Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

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