PL

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EN

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: disodium edetate (EDTA 17%), water Ph. Eur., excipients.

INTENDED USE
ENDO-SOLUTION is intended for mechanical root canal preparation. The product facilitates preparation and cleaning of the canal, removes the smear layer and exposes dentin tubule orifices. By flushing out calcium and magnesium ions, disodium edetate (EDTA) softens dentine surface layer to allow its easier removal and restoration of the canal.

Adapter with a plug for taking the liquid preparations into the syringe.

- Unscrew the bottle cap.
- Screw the adapter on the bottle.
- Remove the blue plug, screw the end of the syringe into the adapter's opening. Turn the bottle upside down and pull the liquid into the syringe, while simultaneously holding the adapter. Next, put the bottle upright on the table top and gently unscrew the syringe from the adapter. Use luer-lock syringes with a rubber plunger.
- After each use, close the adapter's opening with a blue plug.
- For transport, remove the adapter and close the bottle with a bottle cap.

INSTRUCTIONS FOR USE
To prepare the root canal, insert 2-3 drops of liquid ENDO-SOLUTION into the canal while preparing it mechanically for approx. 30 seconds. Rinse with physiological saline. Repeat a few times. Rinsing with sodium hypochlorite (CHLORAXID) is recommended.

CONTRAINDICATIONS
Do not use for patients who are hypersensitive to the product components. Do not leave ENDO-PREP CREAM in the canal or the tooth cavity between visits. This can lead to excessive softening of dentin.

PRECAUTIONS
Extreme caution is advised when perforation is present. May cause severe skin burns and eye damage. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting. Drink plenty of water. Seek medical advice.

Use a rubber-dam when working with the product. Use protective gloves, glasses and protective clothing.
DISPENSER IS NOT INTENDED FOR STORING LIQUIDS!
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in original packaging below 25°C.
Keep out of reach of children.
Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Bottle containing 50 ml, 120 ml or 200 g of preparation, adapter with a cap, dispenser.

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