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EN

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: 100% zinc oxide Eu. Ph. Purity.

INTENDED USE

ZINC OXIDE is recommended for making pastes, mainly with EUGENOL, used for temporary filling of cavities and root canals. Mixture of zinc oxide with eugenol as a temporary filling has drying effect and stimulates creation of the secondary dentine.

INSTRUCTIONS FOR USE

1. When making a temporary filling of the defects, the mixture must not stick to the glass container nor to the spatula.
2. When making a temporary filling of root canals, the mixture should have a creamy consistency. Leave the filling in the cavity or in the root canal for a period of 1 week but no longer than 30 days.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product components.

PRECAUTIONS

May cause eye irritation In case of contact, rinse immediately with plenty of water and seek medical advice.

If swallowed, drink plenty of water. Do not induce vomiting. Seek medical advice. Use a rubber-dam when working with the product.

Use protective gloves, glasses and protective clothing. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original packaging below 25°C. Protect from moisture. Keep out of reach of children.

Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Container with 50 g of the preparation.

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