ENDO-ASPIRATOR SMART SYSTEM

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ENDO-ASPIRATOR SMART SYSTEM

Wireless root canal drying system

**PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE
TO BE USED BY A DENTIST ONLY**

THE KIT INCLUDES:

1. Suction device
2. Short hose (2 pcs)
3. Long hose (2 pcs)
4. Bottle adapter (2 pcs)
5. Bottle for liquids (2 pcs)
6. Applicator (10 pcs)
7. Endo-Aspirator PRO
8. Stand
9. Ramrod for cleaning the Endo-Aspirator PRO
10. Micro-USB cable for charging
11. Replacement supplies (inner filter 9 pcs, rubber O-ring 2 pcs)

HANDLING THE ENDO-ASPIRATOR SMART SYSTEM KIT:

Endo-Aspirator PRO is a reusable instrument, while the bottle with the adapter, hoses and applicators are intended for single use.

1. Handling stainless steel instruments

The manufacturing process involves treatments to ensure corrosion resistance but the life of the instruments depends on proper use and maintenance.

Cleaning, rinsing and washing

The instruments are delivered non-sterile. Before the first sterilisation, the factory new instruments are to be removed from the packaging and thoroughly washed in warm water with a detergent. Before each use in a new patient, the instruments must be sterilised.

Sterilisation

All (reusable) instruments must be disinfected, washed to remove the disinfectant and dried before sterilisation. Ultrasonic cleaning is particularly recommended as the most effective and efficient. Instruments should be sterilised in accordance with applicable national requirements. Recommended sterilisation methods:

Method	Minimum parameters
steam autoclave	134°C for 5 minutes

2. Handling the suction device

Protect the device from moisture.

Do not wash the device itself, as it is not waterproof and water may damage it. After use, wipe with a dry cloth.

3. Handling of other kit components

Before treatment, dip the included applicator in spirit (ethyl alcohol) and then connect to Endo-Aspirator Pro.

The applicators, bottle with the adapter and hoses are for single use.

Replace the inner filter with a new one when dirty.

PRECAUTIONS:

Follow the instructions for cleaning and sterilising reusable instruments.

Do not use the device near flammable materials.

The suction device is not waterproof. Do not aspirate liquids directly into the device, do not immerse it in water or wash it under running water or in a dishwasher. Always use the suction device in conjunction with the bottle. Otherwise, the device may be damaged.

Do not try to disassemble or repair the unit by yourself. For repairs, contact the manufacturer.

For charging, use the included USB cable. Make sure that the charging voltage meets the requirements of the device.

To clean the stand, use warm water with a mild detergent (soap, dishwashing liquid) and a soft cloth.

DO NOT use such cleaning agents as alcohols, solvents or acids, as these will tarnish and damage the material.

NOTE! The applicators, bottle with the adapter and hoses are for single use. Reusing the above kit components may pose a risk of secondary infection. The bottle is not intended for storing liquids. After a single use, hand over for disposal to an authorized collector.

STORAGE CONDITIONS:

Store the instruments at 5-30°C and no more than 70% RH. Before every use, check the reusable instruments for potential contamination and damage to the material structure (cracks, bends, splits, delamination). Non-compliance with the above rules considerably reduces the instrument durability and shortens its useful life.

SUCTION DEVICE SPECIFICATION::

Input voltage: 5 V

Operating voltage: 3,7 V

Rated power: 2,2 W

Battery capacity: 500 mAh

This marking indicates that the product should not be disposed of with other household waste. To avoid the harmful effects to the environment and human health resulting from uncontrolled waste disposal, please separate the product from other types of waste and recycle responsibly to promote the reuse of material resources as a permanent practice. Remove the battery before disposing of the device. Make sure the power is off before removing the battery. Dispose of the battery in accordance with applicable laws. Do not dispose of the battery with other waste.