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OPAKOWANIE

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EN
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY
COMPOSITION: aluminium chloride 20%, water Ph. Eur., excipients.
INTENDED USE ALUSTAT FOAM is intended for use during dental treatment to staunch slight gingival bleedings. ALUSTAT FOAM in the product has the vasoconstrictive effect on tiny blood vessels, as a result of which the gingival swelling and slight bleedings decrease.
INSTRUCTIONS FOR USE Immerse the supplied applicator in ethanol before use, then install the applicator on the product syringe. To stop the bleeding: Apply ALUSTAT FOAM directly on the bleeding spot. Rinse after 2 minutes. Repeat if necessary. To retract the tissues: 1. Apply ALUSTAT FOAM into gingival sulcus using the supplied applicator. Rinse after 2 minutes or 2. Insert the retraction cord into gingival sulcus and place ALUSTAT FOAM on the cord. Rinse after 2 minutes. A reusable product with a single-use applicator. Re-using the applicator may cause the risk of reinfection.
CONTRAINDICATIONS Do not use for patients who are hypersensitive to the product components.
PRECAUTIONS Irritating to the mucous membranes of the oral cavity, skin and eyes. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting. Drink plenty of water. Seek medical advice. Use disposable gloves and protective glasses while working with the product. In any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.
STORAGE Keep in original packaging below 25°C. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.
HANDLING THE PRODUCT CONTAINER Empty containers should be disposed of responsibly or returned to the manufacturer.
PACKAGING Syringe 0,8 g with preparation, disposable applicators.
Last update: 14.06.2021/5

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