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EN
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY
<p>COMPOSITION: aluminum chloride 25%, water Ph.Eur., excipients.</p>

INTENDED USE
ALUSTAT GEL is intended for use during dental treatment to staunch slight gingival bleedings.
Aluminum chloride in the product has the vasoconstrictive effect on tiny blood vessels, as a result of which the gingival swelling and slight bleedings decrease.

INSTRUCTIONS FOR USE
Before the treatment, immerse the included applicator in alcohol (ethyl alcohol) and then place it on the syringe. Apply the product directly on the bleeding spot, next rinse it with clean water. Repeat if necessary. Keep on the bleeding site for up to 5 minutes.

Product intended for multiple use except the applicator which is for single use only. Multiple use of the applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS
It is irritating to the mucous membrane of the oral cavity, skin and eyes. In case of contact rinse immediately with copious amount of water and seek medical advice. In case of consumption do not induce vomiting. Drink copious amount of water. Seek medical advice. Use disposal gloves and protective glasses while working with the product.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in original packaging at a temperature below 25°C. Store out of reach of children. Expiry date is placed on the main container. Once the container is open, the expiry date doesn't change if the container is tightly closed after each use.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Syringe of 5 ml or 10 ml, dispenser, connector, disposal applicators.
Last update: 13.08.2021/8

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