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EN
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

COMPOSITION: Calcium hydroxide, hydroxyapatite, barium sulphate, excipients.

INTENDED USE

CALCIPAST is intended for use in dental treatment for temporary root canal filling. Additionally, CALCIPAST may be used for temporary filling of deep cavities during indirect or direct pulp capping. It has re-mineralizing effect on dental tissues thanks to its high content of calcium hydroxide. The product supports restoration of damaged periapical tissues. pH=12,5. Radiopaque.

INSTRUCTIONS FOR USE

Immerse the supplied applicator in ethanol before use, then install the applicator on the product syringe.

For root canal: measure the canal length, adjust the endostop on the applicator to the length 2-3 mm shorter than the measured canal length, insert the paste into the canal, remove excess paste from the canal, close the cavity securely.

For cavities: apply the paste on the bottom of the cavity and close the cavity securely. Leave the filling in the cavity or in the root canal for a period of 1 week but no longer than 30 days.

A reusable product with a single-use applicator. Re-using the applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS

Irritating to the mucous membranes of the oral cavity and skin. The product may cause serious eye injury. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting.

Drink plenty of water. Seek medical advice.

Use a rubber-dam when working with the product.

Use protective gloves, glasses and protective clothing. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Keep in original packaging below 25°C. Keep out of reach of children.

Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

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Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Syringe containing 2,1 g of the preparation, disposable applicators.

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UK
BEFORE USING THE PRODUCT PLEASE READ THE INSTRUCTIONS CAREFULLY

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INTENDED USE
CALCIPAST is intended for use in dental treatment for temporary root canal filling. Additionally, CALCIPAST may be used for temporary filling of deep cavities during indirect or direct pulp capping. It has re-mineralizing effect on dental tissues thanks to its high content of calcium hydroxide. The product supports restoration of damaged periapical tissues. pH=12,5. Radiopaque.

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

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