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EN
<p>BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY</p> <p>PRODUCT FOR DENTAL USE ONLY</p>

COMPOSITION: eucalyptus oil 100%

INTENDED USE
EUCALYPTOL is intended for use during repeated endodontic treatments to remove gutta-percha from the root canal.

INSTRUCTIONS FOR USE
Before the treatment, immerse the attached pipette into ethyl alcohol.

- Using the pipette, insert 1-2 drops of liquid into the root canal.
- Remove gutta-percha.
- Use another dose of preparation if necessary.
- If gutta-percha is still dry or crumbly, apply EUCALYPTOL again.
- Repeat long as needed.

The product is reusable while the pipette is disposable. Multiple use of the pipette may cause the risk of re-infection.

CONTRAINDICATIONS
Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS
It can cause allergic skin reaction. Use rubber-dam, gloves, protective glasses and protective clothing while working with the product. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in original packaging below 25°C. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Bottle with 10 ml of preparation, application pipettes.

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