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EN

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

INGREDIENTS: Orange oil 100%

INTENDED USE

ORANĖE GUTTANE is intended for use during repeated root canal treatments to remove gutta-percha cones from the root canal.

INSTRUCTIONS FOR USE

Before starting the procedure, dip the pipette (shipped with the product) in ethyl alcohol.

- Use the pipette, put 1- 2 drops of liquid in the root canal.
- Remove the gutta-percha with the selected root canal instrument.
- If necessary, use another dose.
- Re-apply ORANĖE GUTTANE if gutta-percha is dry or crumbly.
- Repeat the use until the desired effect is reached.

The product is reusable, while the pipette is disposable. If re-used, the pipette may create a risk of secondary infection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product’s components.

PRECAUTIONS

Skin irritant. In case of contact, wash with soap and water and rinse thoroughly.

Irritating to the mucous membranes of the mouth. In case of contact rinse with water. It can cause allergic skin reaction.

In case of eye contact, rinse the opened eye for several minutes under running water and consult a doctor.

If swallowed, drink plenty of water and seek medical advice.

Use rubber-dam, gloves, protective gloves and clothing while working with the product.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original packaging at a temperature below 25°C.

Protect from sunlight.

Keep out of reach of children.

Expiry date is placed on the main container. Once the container is open, the expiry date doesn’t change if it the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle of 10 ml of preparation, application pipettes.

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