ENDO  aspirator

ENDO aspirator **PRO**

PL

Spis treści

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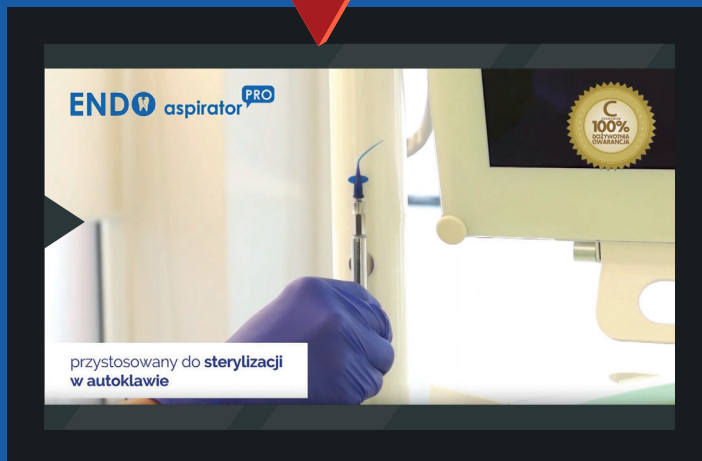
EN

Table of Contents

1. General Information.....	10
2. Instructions for Use.....	12
3. Tool attachment instructions.....	13

ENDO aspirator^{PRO}

for removal of liquids from root canals

1. Handle
2. Applicator
3. Luer Lock hose
4. Connector
5. Magnet
6. Ramrod for cleaning

ENDO-ASPIRATOR PRO

EN

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY

TO BE USED BY DENTIST ONLY
NON - STERILE PRODUCT

HANDLING PRODUCTS MADE OF STAINLESS STEEL:

The ENDO-ASPIRATOR PRO handle is made of stainless steel. Special processes have been undertaken during the manufacture of the product to ensure corrosion-resistance. In spite of this, the durability of this product depends on its proper use and correct maintenance.

HANDLING OF ENDO-ASPIRATOR PRO:

The ENDO-ASPIRATOR PRO handle is made of stainless steel and is designed for multiple use, while the included hoses and applicators are for single use only. Before each use, product should be inspected for contamination, damage to the material structure (cracks, bends, splits, scratches – deeper scratches). Failure to adhere to the above stated instructions decreases product durability and reduces product lifetime.

CLEANING, RINSING AND WASHING OF:

The packaged product is non – sterile. Endo-Aspirator PRO handle that is brand new should be thoroughly cleaned with a non-scratching soft bristle brush and an instrument cleaning solution (detergent: ANIOSYME X3; solution prepared according to the manufacturer's instructions). Ultrasonic cleaning is also recommended. After cleaning, product should be rinsed with purified water and dried, as moisture can adversely affect the effects of sterilisation. Product must be cleaned, rinsed, dried again and sterilised before each use on a subsequent patient.

STERILISATION OF ENDO-ASPIRATOR PRO:

ENDO-ASPIRATOR PRO must be disinfected, then washed out and then dried before sterilisation. Only the handle is sterilized. Do not autoclave a handle with a hose or applicator attached. Product should be sterilized in accordance with national requirements.

Recommended sterilisation methods:

Method	Minimal parameters:
Steam autoclave	134°C, for 5 minutes
	121°C, for 20 minutes

PRECAUTIONS

Adhere to the guidelines for cleaning and sterilization of the dental instruments.

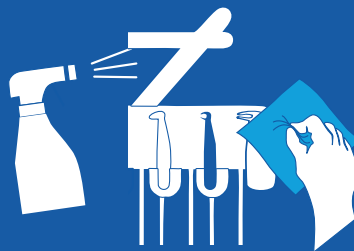
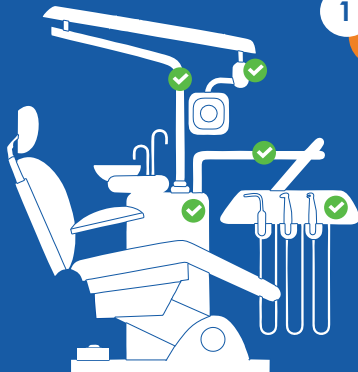
STORAGE

Product should be stored in temperatures ranging between 5-30°C and relative humidity below 70%. Failure to adhere to the above stated instructions decreases product durability and reduces product lifetime.

Tool attachment instructions.

1

Choose any flat surface to mount the Endo-Aspirator PRO on a unit or a microscope.

2

Clean the tool attachment area with any degreasing agent.

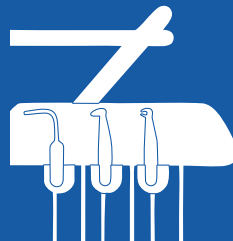
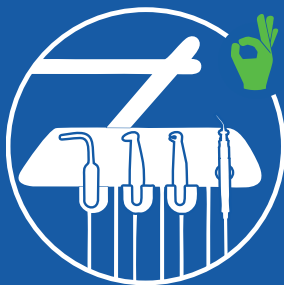
3

Remove the protective layer from the magnet sticker.

4

Apply the adhesive magnet layer to the desired place and press down the magnet.

5

Place the Endo-Aspirator PRO handle to the magnet.

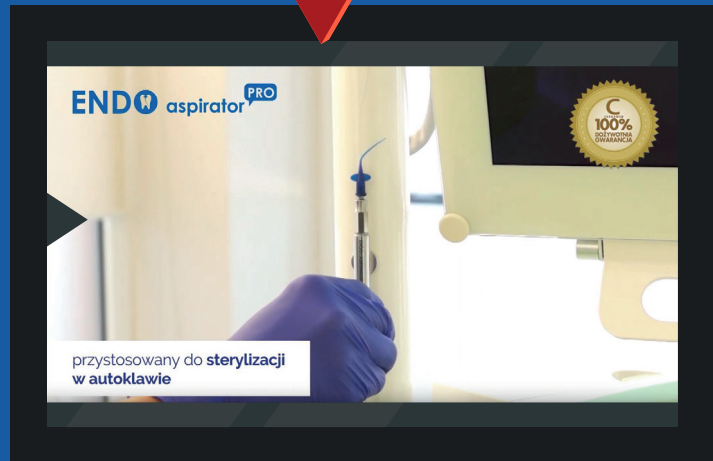
Examples of mounting Endo-Aspirator Pro:

Scan the code and see the video

about how to prepare the Endo-Aspirator PRO for work

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