

PL

SMART SHIELD

Ostonka pod okulary



INSTRUKCJA UŻYCIA WYROBU

PRZEZNACZENIE

Ostonka zmniejsza ryzyko związane z zakażeniami drogą kropelkową, chroni przed działaniem mało szkodliwych czynników mechanicznych.

SPOSÓB MOCOWANIA

Delikatnie wyjąć folię, unikając zabrudzenia i zarysowania.

Umieścić arkusz folii na okularach za pomocą zamocowanych klipsów.

Mocować folię, gdy okulary są rozłożone.

Odchylenie folii można regulować indywidualnie.

Nie należy składać okularów z zamocowaną folią, przed złożeniem folię należy wypiąć.

KONSERWACJA

Po zabrudzeniu wypiąć folię z okularów, myć w ciepłej wodzie z dodatkiem detergentu, spłukać i pozostawić do wyschnięcia. Nie wycierać. Pozwala to na zachowanie dobrej przejrzystości ostionki. Folię można użyć ponownie.

W przypadku pojawienia się na folii zarysowań lub zmatowienia, będącego wynikiem zużycia – folię należy wyrzucić i użyć kolejnego arkusza.

Producent nie ponosi odpowiedzialności za konsekwencje użycia wyrobu niezgodnie z instrukcją i przeznaczeniem.

OPAKOWANIE ZAWIERA:

-zestaw folii (2 sztuki) z zamocowanymi klipsami do zamocowania na okularach.

EN

SMART SHIELD

Clip on glasses face shield



INSTRUCTIONS FOR USE

INTENDED USE

The shield reduces the risk associated with droplet infections and protects against minor harmful mechanical effects.

METHOD OF ASSEMBLY

Gently remove the foil, avoiding scratching and soiling the product.

Mount the foil sheet on the glasses with the attached clips.

Attach the foil when the glasses are open.

Adjust the foil to a comfortable individual tilt.

Do not fold the glasses with the foil attached – remove the foil first.

MAINTENANCE

If soiled, remove the foil from the glasses, wash in warm water with detergent, rinse and leave to dry. Do not wipe. This will allow the foil to stay transparent. Foil is reusable.

If the foil is scratched or tarnished appear because of wear, discard it and use another sheet.

The manufacturer is not responsible for improper use of the product.

CONTENTS OF SET:

- foil set (2 pieces) with attachment clips.

