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EN	Materiał for making pastes
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY TO BE USED BY DENTIST ONLY	

COMPOSITION: 100% zinc oxide Eo. Ph. Purity.

INTENDED USE

ZINC OXIDE is recommended for making pastes, mainly with EUGENOL, used for temporary filling of cavities and root canals. Mixture of zinc oxide with eugenol as a temporary filling has drying effect and stimulates creation of the secondary dentine.

INSTRUCTIONS FOR USE

ZINC OXIDE and eugenol should be combined in proportions allowing obtaining a paste of the desired consistency. The recommended ratio is one teaspoon of ZINC OXIDE and one drop of eugenol (weight ratio 3:1) using the accessories shipped with CERKAMED products - EUGENOL and ZINC OXIDE.

Leave the filling in the cavity or in the root canal for a period of 1 week but no longer than 30 days.

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CONTRAINDICATIONS

Do not use the product in patients with hypersensitivity to the oxide-zinc-eugenol (eugenol) component of the paste.

PRECAUTIONS

The temporary dental treatment may involve temporary pain. The use of temporary materials may be associated with microleakage. ZOE paste binds in a moist environment, do not over-dry the root canal / cavity before applying the paste. Do not use ZOE paste under composites, as eugenol can interfere with the polymerisation process. ZOE paste has cytotoxic potential. Use cofferdam, disposable gloves, safety goggles/goggles and protective clothing when handling the product. In case of contact, rinse immediately with plenty of water. If swallowed, rinse mouth with water. Do not induce vomiting. Avoid creating and inhaling dust. In case of worrying symptoms consult a doctor. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original packaging below 25°C. Protect from moisture. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Container with 50 g of the preparation.

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